
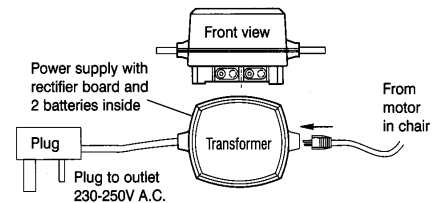


# nightingale

## Bariatric Rise/Recline Patient Chair Operating Instructions

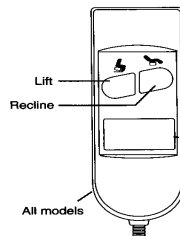

### Chair Instructions

Whilst standing next to the chair, press the up button on the control switch to raise the chair to a height you can sit against. This height will vary from person to person depending on the individual's needs or preference. When the chair is at the desired height, lean against the chair seat and press the down button on the control switch. As the chair continues downward move yourself into a sitting position. Release the button on the switch when the chair is firmly set on the floor.

To recline, continue to press the down button and lean back as the chair reclines. You can control the reclining movement with the switch by releasing the button when you reach the desired position.

To stand up press the up button. If you are in a reclined position the chair will move to a seated position, then slowly and gently begin to rise. As the chair slowly rises gradually shift your weight forward. You can steady yourself by holding the arms of the chair as it rises with you. When the chair reaches a height that enables you to stand easily, release the switch. You may leave the chair in this position as it should be the same height you need to lower yourself to a sitting position.

### Chair Location

Ensure the back of the chair is at least 14 inches (36cm) away from a wall or other obstruction to allow the chair to recline.

Do not allow any objects to restrict the movement of the footrest either up or down.

The chair is fitted with castors to enable easy movement, ensure brakes at the rear are applied before use. Ensure back is slightly reclined before moving, this gives the castors full contact with the floor.

Nightingale Care Beds Ltd  
Unit 1 Ash Road South Wrexham Industrial Estate Wrexham LL13 9UG  
Tel: 01978 660810 Fax: 01978 660828  
Email: [info@nightingalebeds.co.uk](mailto:info@nightingalebeds.co.uk) Web: [www.nightingalebeds.co.uk](http://www.nightingalebeds.co.uk)



# nightingale

## Bariatric Rise/Recline Patient Chair Operating Instructions

### Helpful hints

Continuous operation of the chair will overheat the motor and trip the thermal overload switch. Give the chair a rest between uses. If you run the chair 5-6 full cycles the motor will shut off. If this happens, wait 10-15 minutes the overload switch will reset and the chair will operate again.

It is important to allow the chair mechanism to do the work, do not push back on the chair whilst sitting as this can permanently damage the frame.

The chair is designed for one occupant only, do not allow anyone to sit on your lap, or on the arm, or footrest of the chair

**DO NOT ALLOW CHILDREN OR PETS TO PLAY NEAR THE CHAIR WHILST OPERATING**

**DO NOT ALLOW CHILDREN TO PLAY WITH OR OPERATE THE CHAIR AS THEY MAY SERIOUSLY INJURE THEMSELVES OR OTHERS.**

Ensure that the power cord and transformer are kept together if the chair is moved, any losses will be charged for.

### Important Safety Information

1. Before operating the recliner mechanism, always **ensure that there is nothing to obstruct the free movement of the recliner chair and, in particular, make sure that no children or pets could be trapped in the front part of the recliner when this is closing.**
2. Also **ensure that no surplus wire is trapped inside the chair mechanism before operating the recliner action.**
3. To avoid the possibility of damage, always ensure that the plug is removed from the wall socket before relocating the chair.
4. Do not handle any of the electrical parts, with the exception of the hand control unit, without first switching off the power supply at the wall socket.
5. Do not allow foreign objects to come into contact with the lift or recline mechanism.
6. Do not allow liquids to come into contact with the Hand Controller or other electrical components.
7. Never let children play or operate the chair. This is not a toy but a medical product.

Nightingale Care Beds Ltd  
Unit 1 Ash Road South Wrexham Industrial Estate Wrexham LL13 9UG  
Tel: 01978 660810 Fax: 01978 660828  
Email: [info@nightingalebeds.co.uk](mailto:info@nightingalebeds.co.uk) Web: [www.nightingalebeds.co.uk](http://www.nightingalebeds.co.uk)

